

# How Ear Gear Works



## How To Put Ear Gear On a BTE Hearing Aid



**1.** Remove the earmold together with its tubing from the earhook, leaving the earhook attached to the behind-the-ear (BTE) aid.

**2-3.** Insert the BTE earhook through the end of the Ear Gear with the larger opening and slip Ear Gear over the body of the BTE until the earhook exits through the smaller end of the Ear Gear sleeve.

**4.** Reattach the earmold and tubing to the BTE earhook.

If the Ear Gear Original is a corded model, attach the clip to clothing

## How To Put Ear Gear On a BTE Hearing Aid/ With FM Attachment



1. Remove the earmold together with its tubing from the earhook, leaving the earhook attached to the behind-the-ear (BTE) aid.



2. Insert the BTE earhook through the end of the Ear Gear with the larger opening and slip Ear Gear over the body of the BTE until the earhook exits through the smaller end of the Ear Gear sleeve. Make sure the FM attachment is completely covered by Ear Gear.



3. Reattach the earmold and tubing to the BTE earhook.

If the Ear Gear FM is a corded model, attach the clip to clothing.

## How To Put Ear Gear On A Cochlear Processor



1. Detach the coil from the Cochlear Processor.



2. Insert the ear hook end of the Cochlear Processor through the long opening at the back of the Ear Gear sleeve and then through the small opening at the top of the Ear Gear sleeve.



3. Slip Ear Gear over the body of the Processor until the bottom of the Cochlear Processor is securely encased by the Ear Gear sleeve, and reattach the coil to the processor.

Reattach coil to processor, attach the clip to clothing.

## Ear Gear Mini Curved and Mini Triangle installation method 1



**1-2.** Some models of the mini style of hearing instrument have a modified type of earmold called an ear bud. This ear bud has a detachable tip which can be removed.

**3.** When the tip has been removed slide the tube with the remainder of the ear but through the large opening in the bottom of Ear Gear and feed it out of the small opening in the top.

**4.** Reattach the tip of the ear bud.

If the Ear Gear Mini is a corded model, attach the clip to clothing.

## Ear Gear Mini Curved and Mini Triangle installation method 2



**1-2.** Other models of the mini style of hearing instrument have a dome that can be detached.

**3.** When the dome has been detached then you slip Ear Gear over the hearing instrument until the top of it is fully exposed.



4. Reattach the dome.



5. Slide the hearing aid back inside the Ear Gear until the dome is not visible. Attach the clip to clothing.

### Ear Gear Mini Square installation



The Ear Gear Mini Square is used when there is difficulty removing the earmold or earbud and a reverse style of installation is required. The Ear Gear Mini Square has a larger opening in the top of the sleeve which allows for hearing instruments to be slipped in to the sleeve without removal the mold

If the Ear Gear Mini is a corded model, attach the clip to clothing.